INgezim® Paratuberculosis

R.12.PTB.K1

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INgezim® Paratuberculosis

R.12.PTB.K1

INgezim® Paratuberculosis is an immunoenzymatic assay based on an indirect ELISA technique, which uses a monoclonal antibody (MAb) specific to ruminant IgG and protoplasmic cell extract of *Mycobacterium avium ssp.*

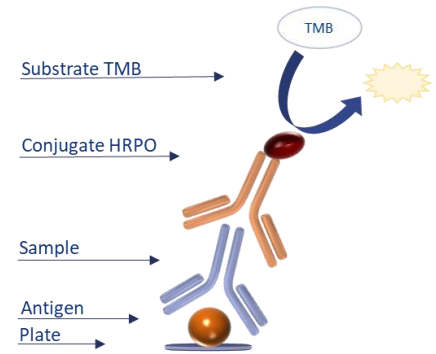
KIT FEATURES

APPLICATION

Detection of specific antibodies to *Mycobacterium avium ssp paratuberculosis* in serum and milk ruminant samples.

TECHNICAL BASE

- Plates are supplied coated with MAP positive antigen. Samples are added to the wells and incubated.
- If the samples contain specific antibodies to *M.avium paratuberculosis*, they will bind to the antigen.
- When the conjugate (monoclonal antibody specific to ruminant IgG, conjugated with peroxidase, AcM-PO) is added, it will bind to the antibodies bound to the antigen. This binding is detected by the development of a colorimetric reaction after the addition of the substrate.

RESULTS INTERPRETATION

The assay establishes two cut off, which will classify the samples as **Positive** or **Negative**, depending on the value of the optical density of the sample in the assay, considering a range of optical densities close to the cut off as **Doubtful** results.

ASSAY VALIDATION

1. Analytical sensitivity.

2 paratuberculosis (PTB) positive reference sera¹ were analysed (bovine and goat). The obtained results indicated that the assay could detect these sera as positive at 1/16 dilution for bovine and 1/32 dilution for goat.

2. Analytical specificity.

There is not cross-reaction with antibodies specific of other related agents.

3. Diagnostic specificity.

Studies with milk. 397 milk samples classified as PTB negative by another commercial ELISA assay available on the market, were analysed. The obtained results indicated **98% relative specificity** compared to this assay.

Studies with sera. 144 serum samples classified as PTB negative by another commercial ELISA assay available on the market, were analysed. The obtained results indicated **100% relative specificity** compared to this assay.

4. Diagnostic sensitivity

Studies with milk. 356 milk samples classified as PTB positive by another commercial ELISA assay available on the market, were analysed. The obtained results indicated **91% relative sensitivity** compared to this assay.

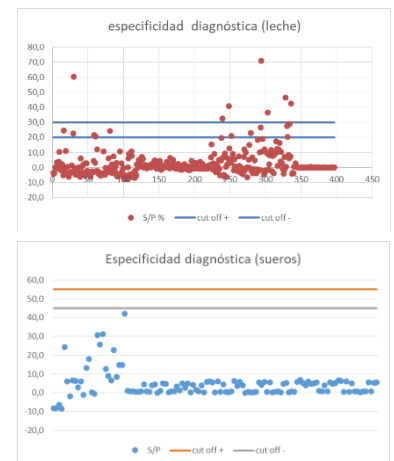
Studies with sera. 110 serum samples classified as PTB positive by another commercial ELISA assay available on the market, were analysed. The obtained results indicated **88% relative sensitivity** compared to this assay.

5. External validation

92 field animal serum samples from China were analysed, which had been previously classified as positive or negative samples by another commercial ELISA assay available on the market. The obtained results indicated 96.7% correspondence.

The assay participated in GD's 2020 Ring Test, obtaining 100% correspondence with the expected results.

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KIT COMPOSITION

- 96 wells microtitration plates
- Vials with Positive Control
- Vials with Negative Control
- Vials with Conjugate
- Bottles with Washing Solution
- Bottles with Diluent
- Bottles with substrate (TMB)
- Bottles with Stop Solution

EXPIRATION: 18 MONTHS
Stored at 2°C-8°C

GOLD STANDARD DIAGNOSTICS MADRID, S.A.
C/Hermanos García Noblejas 39, 8º
28037 MADRID (SPAIN)
Tel: (+34)91 3680501
<http://www.goldstandarddiagnostics.com/>

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¹Allied Monitor Inc (Fayette, MO 65248, USA). Uno de ellos bovino (66-1001 # B132) y otro caprino (66-2001 # C7).